

## Operating Instructions - Timer Mode



1. Press Power button
2. Press + or - buttons to select **timer mode**
  - "10" = 10 minutes on, then turns off.
  - "20" = 20 minutes on, then turns off.
  - "30" = 30 minutes on, then turns off.
  - "P3" = 30 minutes on, and 30 minutes off, continuously.  
Unit must be powered off manually.
  - "P6" = 30 minutes on and 60 minutes off, continuously.  
Unit must be powered off manually.
3. Press COOL or HEAT to select comfort mode.
4. Press "+" or "-" button to select comfort level. (See chart below for conversion of comfort level to temperature in degrees Fahrenheit).

\*\*\* If no timer mode is selected, the ThermaZone Unit will run continuously.

## Manual Operating Instructions - Continuous Mode

Press Power button

Press COOL or HEAT button

Press + or – to select desired comfort mode.

Unit will run continuously until powered off manually.

**\* Please Note:**

Always turn off the ThermaZone Unit when not applied to a patient.

There is no need to disconnect the tubing or pad from machine. If you do disconnect, check the water level when reapplying and add distilled water if needed.

**⚠ WARNING:** This document is not a replacement for the "Instruction and Safety Manual for Your Thermal Therapy Unit"(the "Manual"). Please read the Manual before using ThermaZone products. Failure to follow the instruction in the Manual may result in injury. Keep the Manual in a safe location for future reference.


THERMAZONE COOL LEVEL										THERMAZONE HEAT LEVEL										
10	9	8	7	6	5	4	3	2	1	1	2	3	4	5	6	7	8	9	10	
34°	37°	40°	43°	46°	50°	54°	58°	62°	66°	72° F	80°	85°	90°	95°	100°	105°	110°	115°	120°	125°

⚠ Warning: Review the instruction manual before use. Temperature settings are based on an ambient room temperature of 72°Fahrenheit ( 22.2°C ). Temperatures given are approximate and can vary slightly depending on the environmental conditions. Variations in ambient temperature can increase or decrease the actual temperature output.

# Key Trouble Shooting Issues


## 1 Ensure the ThermaZone Unit has sufficient distilled water.

- Lift the handle.
- Gently peel back black rubber fill cap.
- Slowly pour **distilled water** into the reservoir under the fill cap (water should drain through hole in the center of the reservoir).
- Ensure you have filled between “MIN” and “MAX” levels on the water level indicator to maintain optimal performance.

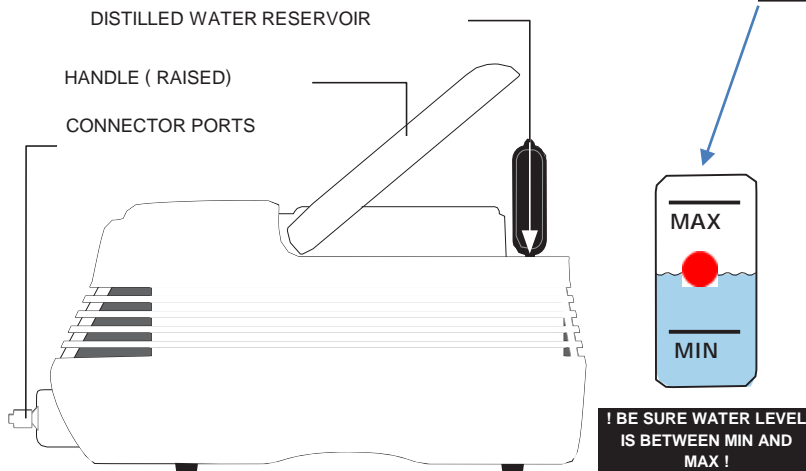

### Refill the reservoir

- Water poured in the reservoir will circulate into the pad.
- Slowly pour additional **distilled water** into the reservoir while the unit is still on.
- Monitor water fill level over the next minute to make sure the water level settles between the “MIN” and “MAX” lines.
- If the red ball in the reservoir is not floating gently tap to release or tilt the unit side to side.

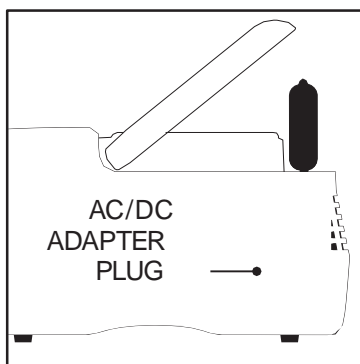

## 2 Ensure ThermaZone Power Supply is being used with the Unit.

- Plug one end of the power cord into the AC/DC adapter plug on the unit. Plug the other end into the electrical outlet. (the control panel will illuminate).
- Press the on/off button (the fan and pump will **start** and the digital display will now say (on) and panel light up.

## 3 Before applying the Pad to a patient, ensure distilled water is circulating.

- Lay pad flat and allow it to come up to temperature before applying it to the patient.
- Make sure that water pathway is not blocked or creased.
- Do not apply too tight. Water pathway can be blocked or compromised. Do not bend the knee when using the knee pad.



Refer to the ThermaZone Safety & Instruction Manual provided with the device.